


You Tube Sownload on the Coogle play

Maillefer Instruments Holding Sàrl Chemin du Verger 3 - 1338 Ballaigues - Switzerland Phone: +41 (0)21 843 92 92 info@dentsplymaillefer.com www.dentsplymaillefer.com

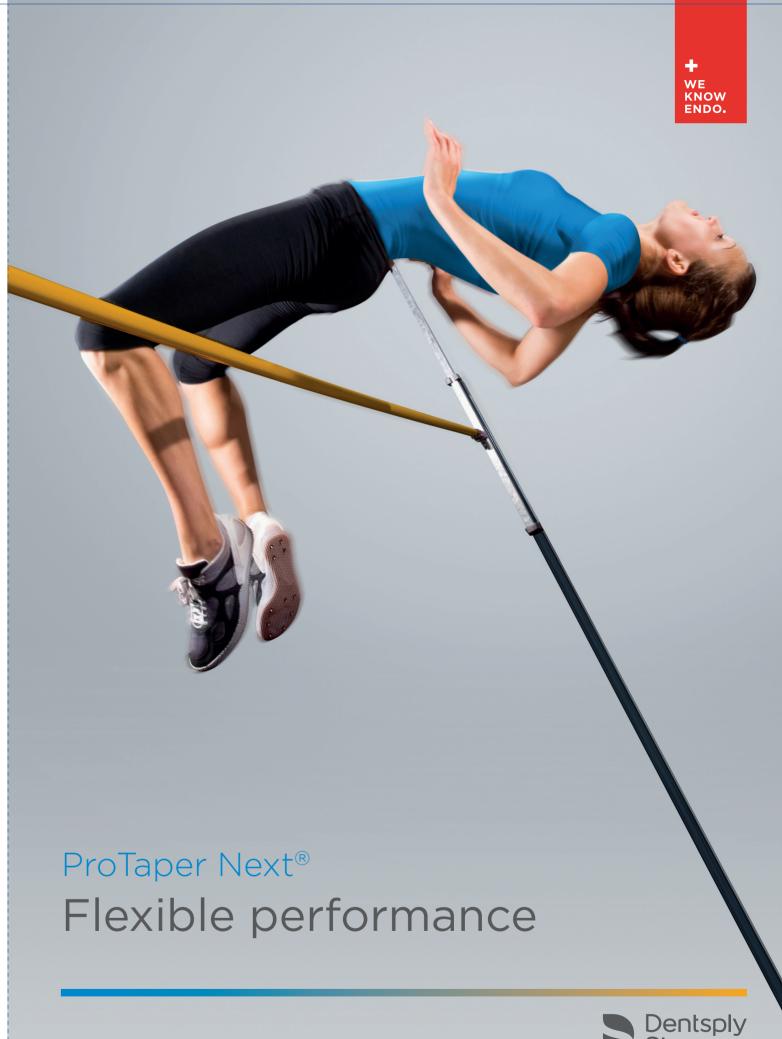









THE ProTaper® PHILOSOPHY REMAINS

A SINGLE SEQUENCE FOR ALL CLINICAL CASES


PROGRESSIVE TAPERS FOR OPTIMIZED CROWN-DOWN TECHNIQUE

COMMONLY APPROVED APICAL FINISHING DIAMETERS

The next generation of the endo reference

ProTaper Next® is the successor to the ProTaper® Universal system, which has been the gold standard in endodontics for many years. It is an efficient solution for endo practitioners seeking a versatile, flexible system that will handle the vast majority of root canal treatments.

• 2001 ProTaper®

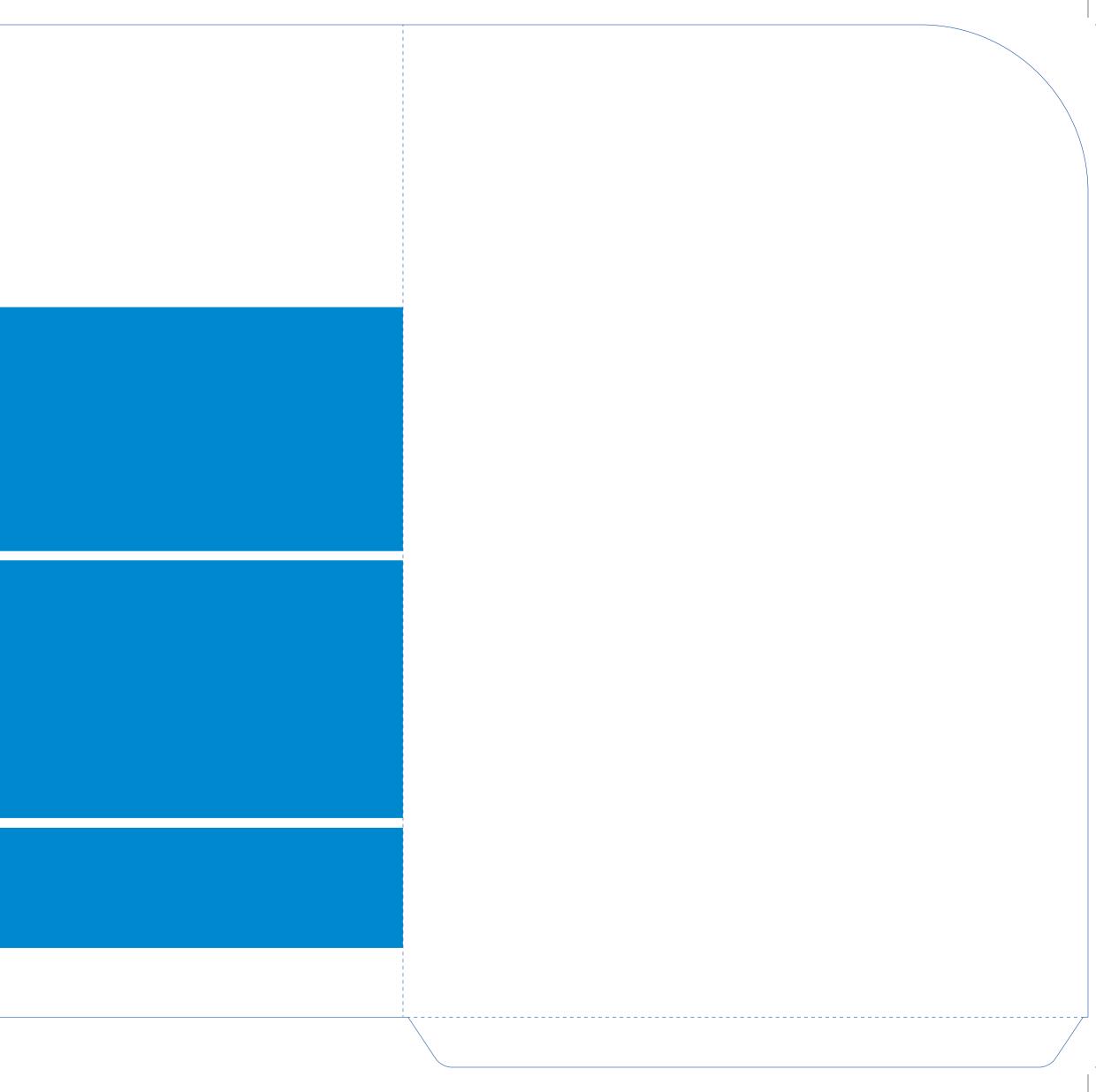

The next generation...

...covers more difficult clinical cases*

The unique swaggering movement and the greater flexibility of the ProTaper Next® files make it possible to shape more severely curved narrow canals than what was possible before with most NiTi systems on the market.

...brings improved safety*


The risk of file breakage is significantly decreased, at the same time; respect of the original root canal anatomy is greatly improved. Instrument and patient safety is essential to good endodontics.

...shortens the shaping time*

A shorter clinical sequence means that less time is spent changing instruments. The high cutting efficiency also reduces the shaping time. For the dentist, this is valuable time that can be used for other procedures, such as irrigation.

...improves debris removal

The unique off-centred rectangular cross section gives the file a snake-like swaggering movement and provides enlarged space for the debris.

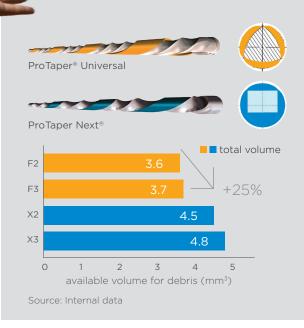

Next generation technology

Swaggering Effect

ProTaper Next®'s innovative off-centred rectangular cross section gives the file a snake-like "swaggering" movement as it moves through the root canal. The rotation of the off-centred cross section generates enlarged space for debris hauling. Optimisation of canal tracking is also achieved due to the "swaggering effect".


M-WIRE®

The M-Wire® NiTi material improves file flexibility, while still retaining cutting efficiency. This is achieved through an advanced Dentsply Sirona proprietary thermal treatment process.



2013 ProTaper Next®

2006 ProTaper®Universal

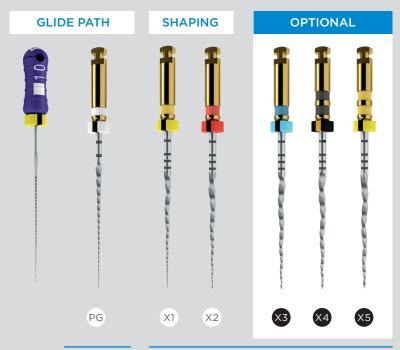

ProTaper Next®

Motor Settings: 300 rpm / 2 - 5.2 Ncm

ProTaper Next® XA (Accessory)

ProGlider[®]

Recommended for preflaring the orifice, removing triangles of dentin, relocating the coronal aspect of a canal away from an external root concavity, and creating more shape as desired.

The next clinical outcome



PRE-OP

Image of maxillary left first premolar. Nonvital pulp had resulted in radiographic lesion of endodontic origin.

XA

POST-OP



Courtesy of Dr. John West

Obturation downpack image demonstrating two canals and five visible portals of exit sealed. ProTaper Next® provides predictable deep shaping preparations for confident and consistent 3D obturation regardless of obturation technique.





Dr. Peet van der Vyver

Dr. Cliff Ruddle

Entering the world of increasing medical standards

Dentsply Sirona Endodontics advocates medical standard care by packaging ProTaper Next® files in pre-sterilised blister packs for single patient use. Using two to three ProTaper Next[®] shaping files for each clinical procedure, instead of four to five previously, makes it affordable for the practitioner to implement a single use policy for endodontic instruments with numerous advantages, namely:



There is no need for reprocessing and re-organising ProTaper Next® NiTi files.



Endodontic files exhibit signs of wear during normal use. They are also subject to cyclic fatigue, which is the main cause of file breakage during patient treatment. This increases dramatically with multiple use. Single use therefore reduces the risk of file breakage and thus increases patient safety.



Optimal cutting efficiency

Further research has shown that using endodontic files for multiple cases actually reduces their cutting efficiency, affecting the performance and satisfaction for both dentist and patient. With ProTaper Next®, there is no reduction in cutting efficiency, as a new instrument is used for each patient.


No risk for cross contamination

No matter how well an instrument is sterilised, research shows that certain bacteria and tissue remnants may remain on the instrument. Additional uses of instruments therefore increase the potential of cross-infection to every subsequent patient.

ProTaper Next®

Ready to go with a complete solution

ProTaper Next[®] is a complete solution, covering all the steps in the endodontic procedure:

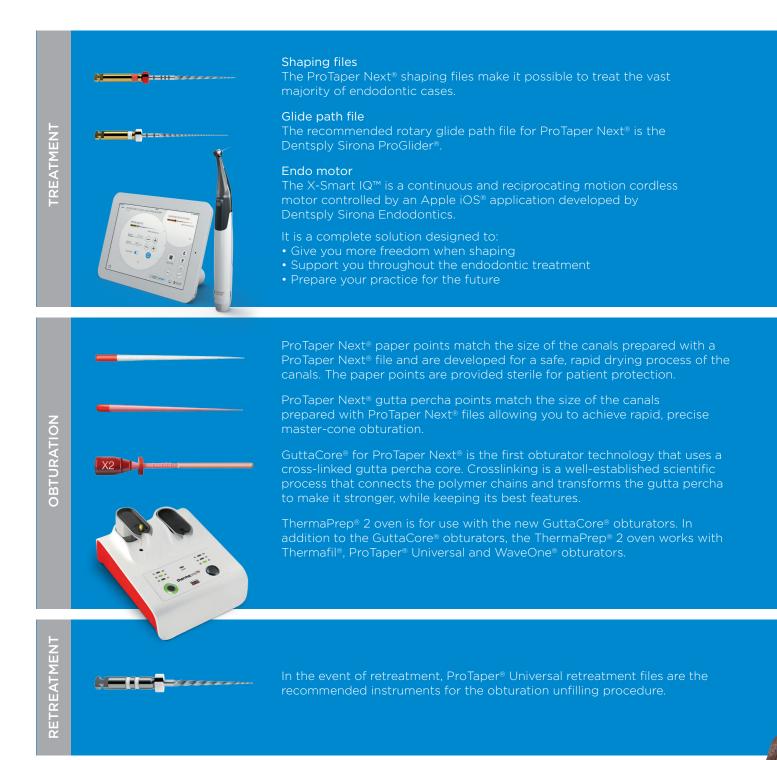

Carl and Carl State

ProTaper Next® A complete solution

PROTAPER • NEXT



Manager Kanger	and the second s		
ProTaper Next®	X1		
A080022100103	File Ster. 21mm	017 .04	6x
A080022500103	File Ster. 25mm	017 .04	6x
A080023100103	File Ster. 31mm	017 .04	6x
A080322100103	File Ster. 21mm	017 .04	3x
A080322500103	File Ster. 25mm	017 .04	3x
A080323100103	File Ster. 31mm	017 .04	3x
ProTaper Next®	X2	H	
A080022100203	File Ster. 21mm	025 .06	6x
A080022500203	File Ster. 25mm	025.06	6x
A080023100203	File Ster. 31mm	025.06	6x
A080322100203	File Ster. 21mm	025.06	3x
A080322500203	File Ster. 25mm	025.06	3x
A080323100203	File Ster. 31mm	025 .06	З×
ProTaper Next®	X3		3
A080022100303	File Ster. 21mm	030 .07	6x
A080022500303	File Ster. 25mm	030.07	6x
A080023100303	File Ster. 31mm	030.07	6x
A080322100303	File Ster. 21mm	030.07	3x
A080322500303	File Ster. 25mm	030 .07	3x
A080323100303	File Ster. 31mm	030.07	3x
A060323100303		030.07	
A060323100303		030.07	
ProTaper Next®			
		040.06	6x
ProTaper Next®	X4		
ProTaper Next® A080022100403	X4 File Ster. 21mm	040 .06	6x
ProTaper Next® A080022100403 A080022500403	X4 File Ster. 21mm File Ster. 25mm	040 .06 040 .06	6x 6x
ProTaper Next® A080022100403 A080022500403 A080023100403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm	040 .06 040 .06 040 .06 040 .06	6x 6x 6x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 21mm	040 .06 040 .06 040 .06 040 .06 040 .06	6x 6x 6x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403 A080322500403 A080323100403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm	040 .06 040 .06 040 .06 040 .06 040 .06 040 .06	6x 6x 6x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403 A080322500403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm	040 .06 040 .06 040 .06 040 .06 040 .06 040 .06	6x 6x 6x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403 A080322500403 A080323100403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm	040 .06 040 .06 040 .06 040 .06 040 .06 040 .06	6x 6x 6x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403 A080322500403 A080323100403 ProTaper Next®	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06	6x 6x 6x 3x 3x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322100403 A080322500403 A080322100403 ProTaper Next® A080022100403	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm X5 File Ster. 21mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06	6x 6x 3x 3x 3x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322100403 A080322500403 A080322500403 A080322100403 A080322100403 A080322100403 A080322100403 A080322100503 A080022100503 A080022500503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm X5 File Ster. 21mm File Ster. 25mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06	6x 6x 3x 3x 3x 3x 6x 6x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322100403 A080322500403 A080323100403 A080022100503 A080022500503 A080023100503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm X5 File Ster. 21mm File Ster. 25mm File Ster. 25mm File Ster. 31mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 6x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322100403 A080322500403 A080322500403 A080322100403 A080322500403 A080322100403 A080323100403 A080022100503 A080022100503 A080023100503 A080322100503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 21mm File Ster. 31mm File Ster. 21mm File Ster. 25mm File Ster. 25mm File Ster. 31mm File Ster. 31mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 6x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A08032100403 A080022100503 A080022100503 A080022100503 A080322100503 A08032200503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 31mm File Ster. 21mm File Ster. 21mm File Ster. 21mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 6x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A08032100403 A080022100503 A080022100503 A080022100503 A080322100503 A08032200503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 31mm File Ster. 21mm File Ster. 21mm File Ster. 21mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 6x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A08032100403 A080022100503 A080022100503 A080022100503 A080322100503 A08032200503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 31mm File Ster. 21mm File Ster. 21mm File Ster. 21mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 3x 3x 3x 3x
ProTaper Next® A080022100403 A080022500403 A080023100403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A080322500403 A08032100403 A080022100503 A080022100503 A080022100503 A080322100503 A08032200503	X4 File Ster. 21mm File Ster. 25mm File Ster. 31mm File Ster. 25mm File Ster. 31mm File Ster. 21mm File Ster. 25mm File Ster. 21mm File Ster. 21mm File Ster. 21mm File Ster. 25mm File Ster. 25mm	040.06 040.06 040.06 040.06 040.06 040.06 040.06 040.06 050.06 050.06 050.06 050.06 050.06 050.06 050.06 050.06	6x 6x 3x 3x 3x 3x 6x 6x 6x 3x 3x 3x 3x

X-Smart IQ™ Motor Handpiece

X-Smart IQ™ Contra-angle

ProTaper Next®	assortment X1 - X3		
A08002219A003	Ass. File Ster.21	×1 - ×3	6x
A08002259A003	Ass. File Ster.25	X1 - X3	6x
A08002319A003	Ass. File Ster.31	X1 - X3	6x
A08032219A003	Ass. File Ster.21	X1 - X3	Зx
A08032259A003	Ass. File Ster.25	X1 - X3	Зx
A08032319A003	Ass. File Ster.31	×1 - ×3	Зx

ProTaper Next®	XA		
A080321900A03 A080021900A03	File Ster. 19mm File Ster. 19mm	019 .035 019 .035	3x 6x
ProGlider®		H	
A092622100103	File Ster. 21mm	016 .02	6x
A092622500103	File Ster. 25mm	016 .02	6x
A092623100103	File Ster. 31mm	016 .02	6x
A092322100103	File Ster. 21mm	016 .02	Зx
A092322500103	File Ster. 25mm	016 .02	Зx
A092323100103	File Ster. 31mm	016 .02	Зx

ProTaper® Universe retreatment D1		NE(8 1	
A141021600103	File Ster.D1	030 .09	6x
A141021800103	File Ster.D2	025.08	6x
A141022200103	File Ster.D3	020 .07	6x
A141220090003	Ass. File Ster.	D1- D3	6x

NiTi Manual adaptors					
A140900090000	Assorted Handle Adapte	ors			6>



A10542000000

Kit contains:

X-Smart IQ™ Motor Handpiece X-Smart IQ™ Contra-angle Endo IQ™ iPad Mini™ Protective Case 10 blisters ProTaper Next® Assortment X1-X3, 25mm (30 files) 1 blister ProGlider®, 25mm (3 files)

A105480000000



Oven

Thermaprep®2 oven A 0176 2300 0200 A 0176 2300 0233 A 0176 1150 0200

EU (230V) UK (230V) (115V)

ProTaper Next® Paper Po A 022Q 00000 200	bint X2 180x sterile paper points
ProTaper Next® Paper Po A 022Q 00000 300	bint X3 180x sterile paper points
ProTaper Next® Paper Po A 022Q 00090 000	bint X2 & X3 90x sterile paper points X2 90x sterile paper points X3
ProTaper Next® Paper Po A 022Q 00090 100	pint X4 & X5 90x sterile paper points X4 90x sterile paper points X5

Gutta Percha Points

ProTaper Next® Gutta Pere A 0220 00000 200	cha Points X2 60x gutta percha points	
ProTaper Next® Gutta Pero A 0220 00000 300	cha Points X3 60x gutta percha points	
ProTaper Next® Gutta Pere A 0220 00090 000	cha Points X2 & X3 30x gutta percha points X2 30x gutta percha points X3	
ProTaper Next® Gutta Percha Points X4 & X5 A 0220 00090 100 30x gutta percha points X4 30x gutta percha points X5		









Obturators Blister

Obtura

X2	Gutta Core® E A 1803B 004
	Gutta Core® E A 1803B 004
X4	Gutta Core® E A 1803B 004
X5	Gutta Core® E A 1803B 004

Blister for PTN X2 25 00 6x Blister for PTN X3 30 00 6x Blister for PTN X4 40 00 6x Blister for PTN X5 50 00 6x

tors Pack	K	
	Gutta Core® Pack for PTN X2 A 1803P 004 25 00	30x (5 blisters)
	Gutta Core® Pack for PTN X3 A 1803P 004 30 00	30x (5 blisters)
	Gutta Core® Pack for PTN X4 A 1803P 004 40 00	30x (5 blisters)



Maillefer Instruments Holding Sàrl Chemin du Verger 3 - 1338 Ballaigues - Switzerland **CE** Phone: +41 (0)21 843 92 92 i www.dentsplymaillefer.com Phone: +41 (0)21 843 92 92 info@dentsplymaillefer.com

