


Tetric[®] N-Collection

A complete nano-optimized restorative system

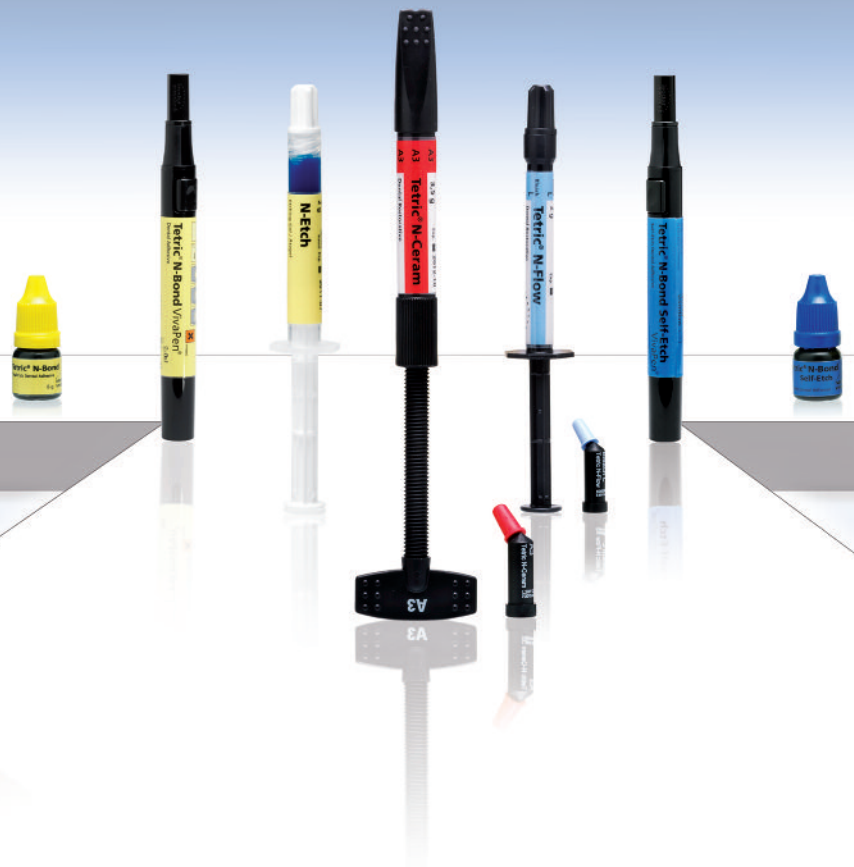

Tetric[®] N-Collection

Tetric[®] N-Ceram | Tetric[®] N-Flow | N-Etch | Tetric[®] N-Bond | Tetric[®] N-Bond Self-Etch

ivoclar
vivadent[®]
passion vision innovation

Tetric® N-

Tetric® N-Bond Self-Etch Nano-optimized self-etching adhesive

Light-curing, self-etching, single-component adhesive for direct restorations

Available in the economical bottle and the convenient VivaPen*

- Consistently high bonding values for a durable bond (>20 MPa)
- Acetone-free chemistry – water is used as the solvent
- Storage at room temperature
- Reduced postoperative sensitivity


*not available in all countries



Easy and time-saving direct application with the VivaPen



Homogeneous adhesive and hybrid layer with well pronounced micro-tags

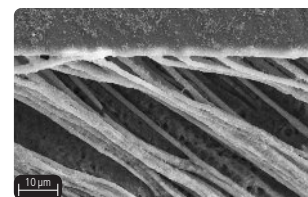

Source: R&D Ivoclar Vivadent AG, Schaan, March 2008

Tetric® N-Bond & N-Etch Nano-optimized bonding system


Light-curing, single-component total-etch adhesive system for direct and indirect restorations

Available in the economical bottle and the convenient VivaPen*

- Consistently high bond strength on enamel and dentin for clinical success
- Nano-fillers ensure a visible glossy coat which indicates complete coverage
- Formation of a homogeneous layer for improved adhesion to the tooth structure
- Reduced postoperative sensitivity
- Acetone-free chemistry – the solvent ethanol provides for easy application in one layer


N-Etch, the 37% phosphoric acid gel, is Tetric N-Bond's best partner for enamel etching and dentin conditioning.

*not available in all countries

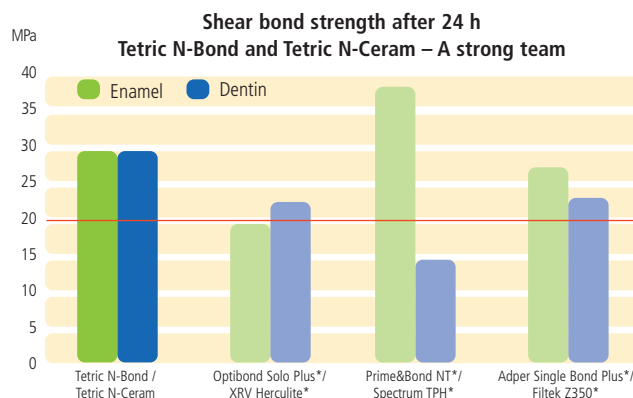

*) not registered trademarks of Ivoclar Vivadent AG
Source: R&D Ivoclar Vivadent AG, Schaan, March 2007

Collection

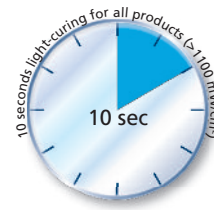

Tetric® N-Flow Nano-optimized flowable composite

Light-curing, radiopaque, flowable nano-hybrid composite

Ideal for use as a cavity liner, for restoring small and Class V cavities and for extended fissure sealing



Available in 10 shades:

A1, A2, A3, A3.5, A4, B2, A3.5 Dentin, T, Bleach L, Bleach I

- The optimum partner for Tetric N-Ceram
- Excellent wetting behaviour allows for convenient application in all areas
- Outstanding stability – ideal for Class V restorations
- High level of radiopacity for a sound diagnosis



Radiopacity of Tetric N-Flow

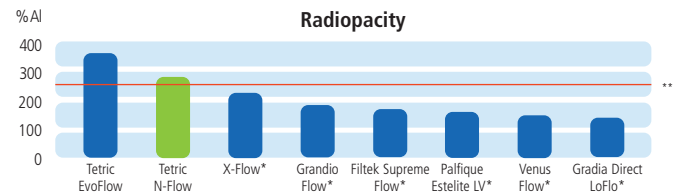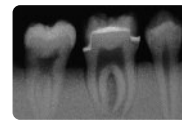

*) not a registered trademark of Ivoclar Vivadent AG
 **) minimum radiopacity as defined by Willems et al. 1991 J. Dent. 19:362-365
 Source: R&D Ivoclar Vivadent AG, Schaan, March 2007

Tetric® N-Ceram Nano-optimized mouldable composite

Light-curing, radiopaque nano-hybrid composite for direct restorative procedures

Universal use – for anterior and posterior restorations


Available in 16 shades:

A1, A2, A3, A3.5, A4, B2, B3, C2, C3, D3, A3.5 Dentin, B2 Dentin, T, Bleach L, Bleach I, Bleach M


- Nano-fillers improve the material's properties:
 - reduced shrinkage and low shrinkage stress
 - high level of radiopacity
 - low wear
 - fast polishability and high gloss
- Nano-colour pigments are responsible for the material's unique chameleon effect and natural esthetic results
- The nano-modifier ensures optimum stability, excellent sculptability and convenient handling properties




SEM images

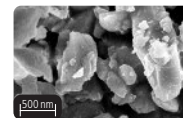


Fillers 700 nm

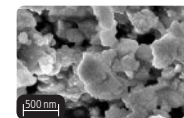

Fillers 500 nm

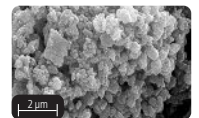

Prepolymers

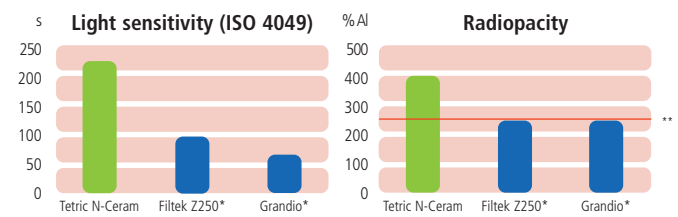

Source: R&D Ivoclar Vivadent AG, Schaan, March 2007

Source: R&D Ivoclar Vivadent AG, Schaan, March 2007

*) not a registered trademark of Ivoclar Vivadent AG
 **) minimum radiopacity as defined by Willems et al. 1991 J. Dent. 19:362-365

Tetric® N-Collection

A complete nano-optimized restorative system

DELIVERY FORMS

Tetric N-Bond Self-Etch

Refill bottle
1 x 6 g bottle

Refill VivaPen
1 x 2 ml VivaPen
20 x VivaPen brush cannulas

Accessories
50 x Vivadent applicator white
100 x VivaPen brush cannulas
500 x VivaPen sleeves

Tetric N-Bond

Refill bottle
1 x 6 g bottle

Refill VivaPen
1 x 2 ml VivaPen
20 x VivaPen brush cannulas

Accessories
50 x Vivadent applicator white
100 x VivaPen brush cannulas
500 x VivaPen sleeves

N-Etch

Refill
2 x 2 g N-Etch syringes
10 x applicator tips

Accessories
20 x applicator tips

Tetric N-Flow

Assortment Pack
8 x 2 g syringes
1 x A1, 4 x A2, 2 x A3,
1 x A3.5
1 shade guide
Various accessories

Refill syringe
1 x 2 g available in the following shades:
A1, A2, A3, A3.5, A4, B2, A3.5 Dentin, T, Bleach L, Bleach I

Refill cavifils
10 x 0.25 g available in the following shades:
A1, A2, A3, A3.5, A4, B2, A3.5 Dentin, T, Bleach L, Bleach I

Accessories
20 x applicator tips

Tetric N-Ceram

Intro Pack with Tetric N-Bond
4 x 3.5 g syringes
1 x A2, 1 x A3, 1 x A3.5,
1 x B2
1 x 6 g Tetric N-Bond
1 x 2 g N-Etch
1 shade guide
Various accessories

Intro Pack with Tetric N-Bond Self-Etch
4 x 3.5 g syringes
1 x A2, 1 x A3, 1 x A3.5,
1 x B2
1 x 6 g Tetric N-Bond Self-Etch
1 shade guide
Various accessories

Refill syringe
1 x 3.5 g available in the following shades:
A1, A2, A3, A3.5, A4, B2, B3, C2, C3, D3, A3.5 Dentin, B2 Dentin, T, Bleach L, Bleach I, Bleach M

Refill cavifils
10 x 0.25 g available in the following shades:
A1, A2, A3, A3.5, A4, B2, B3, C2, C3, D3, A3.5 Dentin, B2 Dentin, T, Bleach L, Bleach I, Bleach M



These products form part of our "Composites" competence area. All the products of this area are optimally coordinated with each other.

Descriptions and data constitute no warranty of attributes.
Printed in Liechtenstein
© Ivoclar Vivadent AG, Schaan/Liechtenstein
607776/0510/e/BVD

Ivoclar Vivadent AG
Bendererstr. 2
FL-9494 Schaan
Principality of Liechtenstein
Phone +423 / 235 35 35
Fax +423 / 235 33 60
www.ivoclarvivadent.com

ivoclar
vivadent
passion vision innovation